MEDSHOP
POLSKA

Lampa LED
Lampa halogenowa

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Cechy produktu :

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Product features

1. Small lamp holder of 75 mm diameter, which does not interfere in the user's field of vision.
2. Flexible cross arm of 700 mm long, which can be positioned in multiple directions.
3. Wheeled base to move the lamp according to the user needs.
4. Specially designed to perform explorations in different medical departments as well as additional lighting in surgical rooms.

Technical specifications

	KS-Q3	S-Q35
Color temperature	(5000±300)K	
Illumination	5000 – 20000 Lux	5000 – 40000 Lux
Temperature rise	≤ 2°C	≤ 8°C
Power source	AC 220V±22V 50Hz±1Hz	
Energy consumption	5VA	50VA
Bulb	LED 3W	Halogen light 12V/35W
Optimal serviceable range	300mm-500mm from lamp holder	

Components

This examination lamp consists of three detachable parts:

1. Flexible arm with lamp holder
2. Vertical stand
3. Wheeled base

The arm with lamp holder and the vertical stand are packed with foam material into a carton.
The wheeled base is packed in a separate carton.

Installation

As shown in the figure, insert the vertical stand (2) into the wheeled base (1).
Then, insert the flexible arm with lamp holder into the vertical stand and tighten the bolt firmly into its correspondent hole.

Operation

1. Moving the lamp: Hold the vertical stand (2) with one hand and move the lamp to the desired position.
2. Lamp holder adjustment: Hold the vertical stand (2) with one hand and the lamp handle (7) with the other hand. Pull the handle to the desired position.
3. Switch off: Fully turn the control knob (4) counterclockwise until hearing a click.
4. Bulb replacement: Unscrew the inferior part of the lamp holder (8) by turning it counter clockwise. Take out the damaged bulb and replace it by a new one. Place again the inferior part of the lamp holder.

Note: Before replacing the bulb, make sure the lamp is unplugged from the mains and wait for the lamp holder to cool down in order to prevent burns.

Maintenance

Clean regularly the lamp structure and the lamp holder glass with a damp cotton cloth and mild detergent.

Safety precautions

- Use a safe single electrical outlet to connect the lamp.
- In order to prevent burns, never touch directly the lamp holder glass during use.

Transport and storage

- Keep the lamp in a well-ventilated room, free of flammable gas and no more than 80% of relative humidity.
- Avoid shock, excessive vibration and moisture during transportation.

Included accessories

- 1 Flexible arm with lamp holder
- 1 Vertical stand
- 1 Wheeled base
- 1 Instructions manual

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MADE IN P.R.C.


Potencia/Power: AC 220V±22V 50Hz±1Hz

LOT: VER CAJA/SEE OUTER BOX

Dystrybutor / Distributed by : Medshop.pl

MINSTON INTERNATIONAL CO., LIMITED
Factory; No.1808 Fenfa Mainsion,
TianShan Road, Shantou ,China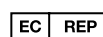

WFD Partners SARL 86 rudes
Bas Rogers 92800 PUTEAUX-France